

This customised splint will put an end to your night-time dental wear. It's the first step towards better smiles throughout the day.

Created especially for you, it is customised to your needs using the latest digital technology.

If you have any questions, consult your dentist.

DIGITAL CUSTOM-MADE MEDICAL DEVICE BY CORUS. HYGIENISED PRODUCT.

NOBRUX HAS BEEN MADE WITH A CLASS IIA BIOCOMPATIBLE MATERIAL COMPLYING WITH THE ISO10993-1 STANDARD FOR CYTOTOXICITY - SENSITISATION AND/OR IRRITATION OF THE ORAL MUCOSA

nobrux

Protection by night, smiles by day



NORBUXINSEN_REV02_230301

Four steps to look after your splint

DURING THE FIRST DAYS OF US

You may encounter the following symptoms:

—
Feeling of pressure.

—
Speech alterations.

—
More salivation than usual.

—
Slight chafing where the splint touches your mouth.

In one week, these symptoms will disappear. If they persist, consult your dentist.

ADVICE

Use lip balm when you're wearing the splint.

—
Use the case, spray and cleaning tablets we provide to keep it in good working order.

—
Follow the user instructions below.

If you look after your splint properly, you'll make sure it keeps working and lasts longer.

1—Protect Splint

While you're sleeping, your splint prevents involuntary tooth wear caused by friction owing to conditions like teeth grinding. It helps relieve pressure and keep your jaw more relaxed.

HOW

Brush your teeth before using your splint. Position it on your dental arch using your fingers to hold it there. Don't bite down until you have it in the right place. To take it out, do the same thing in reverse. In the morning, clean the splint using its own dedicated brush and water. Dry it using a cloth or absorbent paper.

WHEN

Use it every night.
Clean it every morning.

2—Refresh Mint-flavoured spray

For an intense sensation of freshness.

HOW

Spray two or three times directly inside the splint. It's now ready for use.

WHEN

Every night before you put your splint in.

IMPORTANT

At no time should any water in contact with the splint exceed 35°C.

3—Store NobruXBox

This is ideal case to store your splint during the day or to take it with you on a trip.

HOW

Be careful! Open the case by sliding the lid sideways.

Clean your NobruXBox with soap and water using a sterile gauze or brush to stop mould or bacteria forming inside that could attack the splint.

WHEN

Use it every day.
Clean it once a week.

4—Clean Cleaning tablets

The cleaning tablets Deeply sanitises the splint, eliminates unpleasant odours and provides a pleasant minty fresh taste.

HOW

Put a cleaning tablet in a warm glass of water. When it starts bubbling and releases a blue liquid, put the splint in the glass. When the blue colour disappears (10–20 minutes), remove the splint and rinse it thoroughly with water. Finally, dry the splint and put it in its NobruXBox with the inner part of the splint facing downwards.

WHEN

Once a week.

** Failure to follow all the instructions may affect the properties of the splint material and cause its colour and fit to change.*