Patient protocol,
prosthesis cleaning

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More about:

Welcome to your new life with Optima. From today, you will be able to smile, talk and eat with normality and peace of mind. As we want you to be able to do all of this for a long time, we have prepared some simple maintenance instructions that are easy to follow and will keep your Optima in perfect condition.

First, we are going to tell you about some things that you may notice during the first few days of using your Optima which are all completely normal:

- Sensation of pressure.
- Alterations in speech.
- Increased salivation.
- Slight chafing in the support areas of the denture.

Don't worry, these symptoms will usually disappear in a week as your mouth adjusts. If not, you should consult your dentist, who will be able to help you.

Thanks to Optima you will be able to smile again and enjoy your day-to-day life with complete confidence, but to continue enjoying yourself and to keep your mouth healthy, it is advisable to follow these recommendations:

1. Wash your hands with soap and water before starting the process.
2. Remove the prosthesis for a minimum of 4 hours each day to help maintain healthy gum tissues. Perform a thorough clean of the prosthesis and tissues.
3. Rinse your mouth with a special mouthwash for gums.
4. Clean the prosthesis with the Optima brush to remove all accumulated debris using a non-abrasive soap solution. The Optima toothbrush consists of two heads. The larger head is designed to clean the surface of the denture,

especially flat surfaces, and external areas. The smaller brush head is designed to clean the teeth and small internal areas of the prosthesis with maximum precision.

The Optima toothbrush consists of two heads a larger head designed to clean the surface of the denture, especially flat surfaces and external areas. The smaller brush head is designed to clean the teeth and small internal areas of the prosthesis with maximum precision.

A good brushing is the easiest and most efficient way to keep your Optima in perfect condition.

5. After cleaning it, store and protect your prosthesis in water inside the Optima Box. It has been specifically designed to keep it clean and protected all night long. The Optima Box is made from recycled plastic bottles, helping the environment by reducing plastic waste in the world.

The Optima Box takes care of your Optima and the environment, as it is made from recycled plastic from used bottles. This helps to reduce the use of plastics and waste in the world.

Note: if you do not have the Optima Box uses a suitable container.

6. Finally, when you get up, start your day by washing your mouth and tongue before putting your prosthesis back in. Now you can enjoy the day with complete peace of mind.

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