Maintenance guide,
for the Optima prosthesis.

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More about:

Welcome to your new life with Optima.

From today onwards, you can smile, speak, and eat with complete confidence. To ensure Optima serves you well for many years, please follow these guidelines.

What you may notice in the first few days.

During the first week, it is normal to experience:

- A slight sensation of pressure.
- Temporary changes in pronunciation.
- Increased salivation.
- Small sore spots on the gums.

These symptoms usually disappear on their own. If they persist for more than seven days, book an appointment with your dentist.

Daily routine in 6 steps:

Step 1 — Wash your hands with soap and water.

You avoid germs when handling the prosthesis.

Step 2 — Remove the prosthesis for at least 6-8h/day.

This allows the tissues in your mouth to rest.

Step 3 — Rinse your mouth with a gum-care mouthwash.

Keeps your gums and tongue fresh.

Step 4 — Brush the prosthesis with the Optima brush and a non-abrasive toothpaste.

The large head cleans flat surfaces; while the smaller one reaches internal areas and teeth.

Step 5 — Store

it submerged in water inside the Optima Box (stainless steel + bamboo lid).

Durable, 100% recyclable steel and renewable bamboo—protecting both your prosthesis and the planet. (If you don't have one, use a clean container.)

Step 6 — Place

In the morning, gently clean your mouth and tongue, then put your prosthesis in place.

You start the day fresh and confident.

Enjoy your day!

IMPORTANT NOTICE

Optima is a medical device made exclusively for you, based on your prescription and under the supervision of a licensed dental professional.

The prosthesis is tailored to your individual needs and must not be used by anyone else.

Optima should be used exactly as instructed by your dental professional, following a daily routine of hygiene and care.

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