OPTIMA®

Any unauthorised tampering with the Customised Medical Device in accordance with Regulation (EU) 2017/745 may alter its characteristics, which could invalidate the corresponding declaration of conformity.

The product is manufactured as prescribed by a licensed healthcare professional. Any modification of the conditions established by the authorised healthcare professional who issued the prescription may result in deviations from the expected outcome of the custom-made medical device. If you have any questions or concerns, please consult your qualified healthcare professional.

INTENDED PURPOSE

The intended purpose of Optima® is to replace missing natural teeth and adjacent structures in the upper or lower arch. Its use provides structural support to oral tissues, helps maintain the vertical dimension, and provides adequate distribution of chewing forces over the oral mucosa.

This removable prosthesis has been custom-made for a patient based on a preliminary prescription from an authorised healthcare professional.

INDICATIONS

Optima® is intended for adult patients (*) diagnosed with edentulism, which is the partial or total absence of natural teeth.

CONTRAINDICATIONS

Optima® is generally NOT intended for:

- Patients with allergies to acrylic.
- Patients with severe neuromuscular disorders or severe motor problems.
- Patients with hyperreflexia.

Partial dentures for:

- Patients who have remaining teeth with a poor prognosis.
- Patients with severe malocclusions.
- Patients with very advanced bone resorption.

Complete dentures for:

- Patients with severely resorbed bone ridges.
- Patients with xerostomia.
- Patients with severe bruxism.

MODE OF ACTION

For partial dentures, it acts by replacing one or more missing teeth, supported by the remaining teeth and mucosa. It uses metal or acrylic retainers to hold the prosthesis in place and distribute chewing forces. The transmission of occlusal loads is shared among the supporting tissues, helping to maintain the functional balance of the dental arch.

For complete dentures, it replaces all the teeth in one arch in patients who have lost all their teeth. It is supported exclusively by the mucosa and alveolar bone. Retention is based on physical principles such as adhesion between the prosthetic surface and the mucosa, peripheral sealing, and the stabilising action of the patient's oral muscles. The masticatory load is distributed over the mucosa of the residual ridge.

HOW TO USE

The dental professional must ensure that the prosthesis fits the patient's dental profile correctly and complies with the prescribed specifications.

Optima® has been designed and customised for each patient, according to the prescription of an authorised professional and under the responsibility of that professional. The product has been manufactured in accordance with the specific characteristics provided by the professional and is intended for the exclusive use of the patient, in line with their individual requirements and/or needs.

Optima® should be used exactly as directed by a licensed dental professional, following a daily routine of hygiene and care.

Visit your dentist regularly and check the fit and condition of your prosthesis at every appointment.

If you experience any discomfort, contact your dentist for a check-up.

If you are undergoing a new dental procedure, confirm with your dentist beforehand whether the use of Optima® is recommended, as whether any adjustments of the device.

CARE AND MAINTENANCE

Optima® is a non-sterile product. Clean and rinse the device before use. Use a denture brush and non-abrasive toothpaste.

Remove the dentures after each meal and rinse them with water to remove any food debris.

Wash your mouth and tongue before putting the dentures back in.

Clean the dentures at least once a day with a denture brush to remove any accumulated debris. Use a non-abrasive toothpaste.

Remove the dentures at night to allow the oral tissues to rest and prevent infection. It is recommended that the dentures be removed for at least 6 to 8 hours a day.

Rinse your mouth with a special a gum-care mouthwash.

Store the dentures in a clean container with water, designed to keep them clean and protected overnight.

WASTE MANAGEMENT

Consult your healthcare professional before disposing of the product and ensure that it is disposed of in accordance with the applicable regulatory instructions in your region for waste management.

WARNINGS AND PRECAUTIONS

⚠ Please remember to handle Optima® with care.

⚠ DO NOT use Optima® if there are any cracks, fractures or deformities.

⚠ Do not use continuously; allow to rest overnight or for a minimum of 6 hours daily.

⚠ DO NOT use abrasive products, bleach or cleaners not designed for dentures.

⚠ Maintain proper oral hygiene.

⚠ If there is pain, persistent irritation or an unpleasant odour, discontinue use and consult your referring professional.

(*) No studies have been conducted on the use of this product in pregnant women. Consult your healthcare professional to assess the potential risks associated with its use.

NOTIFICATION OF SERIOUS INCIDENTS

If any serious incident related to the use of this medical device occurs, it must be reported immediately to the healthcare professional. A serious incident is considered to be any event that has caused or could have caused a deterioration in health, serious illness or death.

SYMBOL*	TITLE AND DESCRIPTION
	MODEL NUMBER - Indicates the model number/type of a product.
	MANUFACTURING DATE - Indicates when the medical device was manufactured.
	SERIAL NUMBER - Indicates the manufacturer's serial number for identifying a specific medical device.
	CATALOG NUMBER - Indicates the manufacturer's catalog number for identifying the medical device.
	UPPER TEMPERATURE LIMIT - Indicates the maximum safe temperature to which the medical device can be exposed.
	NON-STERILE - Indicates a medical device that has not undergone a sterilisation process.
	DISTRIBUTOR - Indicates the entity distributing the medical device within the regional configuration.
	KEEP AWAY FROM SUNLIGHT - Indicates a medical device requiring protection from light sources.
	MANUFACTURER - Indicates the manufacturer of the medical device.
	REFER TO INSTRUCTIONS FOR USE - Indicates the need for the user to consult the instructions for use.
	CAUTION - Indicates that the current situation requires operator awareness or action to prevent unintended consequences.

*STANDARD: ISO 15223-1, MEDICAL DEVICES: Symbol to be used with the information that the manufacturer will provide.

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